

Structure: The twist-off lancets consist of needle, main body and protective cap.

Specifications:

DV-I-007/25	
Gauge	Color
28G	Transparent Blue

Performance: The device is sterilized by irradiation.

Intended use: The twist-off lancet (use with lancing device) is a single use, sterile medical device, which is designed for use of micro blood sampling puncture to obtain capillary blood samples from the fingertip. The twist-off lancet needs using with lancing device

Intended Users: Healthcare personnel, only when used with single lancing device to obtain blood sample from single patient
Patients (including lay person)

Intended Patient population:

Patient who needs to obtain capillary blood samples from fingertip, with no restriction to gender, weight or race.

Indication: The lancet is intended for people who need capillary blood sampling.

Contraindications: None.

Instructions

1. Unscrew the lancing device cap from the body of the lancing device.
2. Insert a sterile lancet into the lancing device firmly.
3. Twist off the protective cap of the lancet.
4. Carefully screw the lancing device cap back to the lancing device.
5. Adjust the puncture depth.
6. Put the lancing device against the side of the finger and press the release button to prick your fingertip. If blood can't be collected, please adjust the puncture depth according to the instruction of the lancing device, and repeat this step.
7. Apply a drop of blood to test strip.
8. Recap lancet and discard it into an appropriate container.
9. Place the lancing device cap back to the lancing device.



The lancet and lancing device must be compatible with each other. When you use DiViLab twist-off lancet with other manufacturers' lancing devices, please be sure that these products are compatible.



We recommend that you shall consult a health care professional before use.

Storage: The products should be stored in a well-ventilated room and kept away from light, moisture and corrosive gases.

Shelf life: 5 years.

Cautions

1. Read the instructions carefully before use.
2. Use the lancets within the expiry date.
3. Do not use if protective cap is removed or broken previously.
4. Do not store lancet in lancing device, discard lancet after use, do not re-use.
5. Lancet is only for single-use, and re-use of the device could lead to cross-infection and damage of the needle.
6. If the used needle is not recapped, it might cause sharp injury of other person and might cause infection or cross-infection.
7. If the needle is broken and embedded into the finger, please consult a healthcare professional immediately.
8. In the event of any serious adverse event, please report to the manufacturer and the competent authority.

Symbol Descriptions

	Batch code		Caution		Keep away from sunlight
	Sterilized using irradiation		Date of manufacture		This way up
	Do not re-use		Use-by date		Keep dry
	Importer		Manufacturer		Fragile, handle with care
	Medical Device		Read the manual book		Do not re-sterilize
	Do not use if package is damaged and consult instructions for use				



Produced by:

TIANJIN HUAHONG TECHNOLOGY CO., LTD.

A01, Plant B, No. 278, Hangkong Road, Tian Jin Pilot Free Trade Zone (Air Port Industrial Park), Tianjin, 300308, China

For:

NANJING LONG MEDICAL TECHNOLOGY CO., LTD.

Building #4, No. 2, Rixin Road, Jiangning District, Nanjing City, Jiangsu Province, China, 211101

Distributed by:

PT ASTRA KOMPONEN INDONESIA

Jalan Lanbau RT.005 RW.010, Kelurahan Karang Asem Barat, Kecamatan Citeureup, Bogor, Jawa Barat - Indonesia